



**REGULAR  
DISPOSABLE AV LINER**

**Requires Addition of  
Filter and Collection Pouch**







**For Use with the  
COLORADO MODEL AV**

**10 per Pack**

**Hamilton Research, Inc.**  
P.O. Box 2099  
South Hamilton, MA 01982  
(978) 468-4460, (800) 367-0266  
Fax: (978) 468-1359  
Equitainer@aol.com  
<http://www.equitainer.com>

### Instructions for Inserting the Colorado Regular Disposable Liner

1. Place Disposable Liner into AV before adding the final amount of hot water (Fig. 1).
2. Pull the end of the liner through the AV.
3. Fold the top 2 to 3 inches of disposable liner over the top of the AV (Fig. 2).
4. Tape the folded top of disposable liner over the top of the AV (Fig. 3).
5. Make sure the disposable liner is not twisted.
6. Cut the end of the liner at desired length (Fig. 4).
7. Place the filter over the cut end and secure with tape (Fig. 5).
8. Place the WhirlPak® Bag or baby bottle line over the filter and tape securely (Fig. 6).
9. Lubricate the disposable liner and finish preparing the AV as usual.
10. While collecting be sure to hold the collection pouch so that the stallion does not pull the liner out while withdrawing.

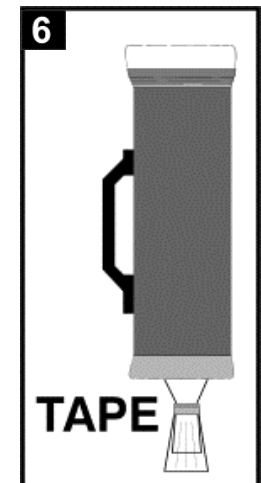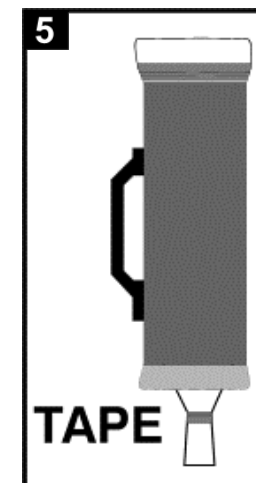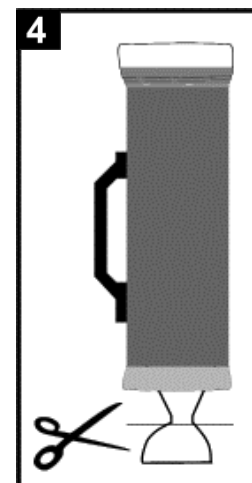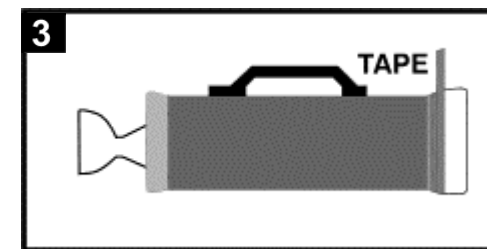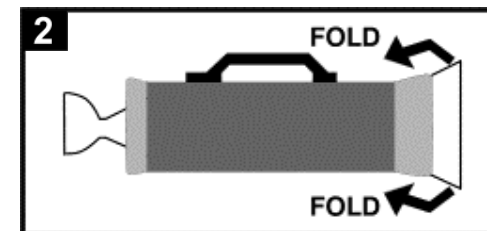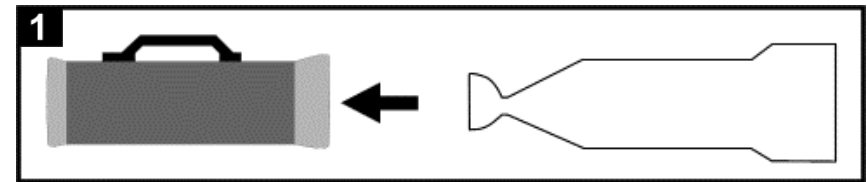

### After Collection

Remove the collection bag from the end of the AV and transfer into a sterile specimen cup.

### Notes

In the Colorado Model AV, Disposable AV Liners take the place of the inner latex collection liner.

In cold weather, store the Disposable AV Liner in an incubator to avoid cold shock the sperm. Also, it is advised that an insulated cover be placed over the collection pouch.